GB 1. Intended use

US **printisol** is a solvent-based insulating agent that forms an insulating layer between the 3D-printed model and the cold-curing resin. For this purpose, high-quality resin models produced using the 3D printing process (e.g. with the Asiga MAX™) should be used.

2. Contraindication

printisol is not suitable for printed models made of thermoplastic filaments and hot polymers. Please test the suitability of **printisol** in individual cases before use.

3. Intended Users

Dental professionals

4. Manufacturing process (fig. 1-7)

1. Carefully block out undercuts on the model with thermal wax.
2. Pour **printisol** directly from the bottle into a container. Close the bottle immediately after use.
3. Apply **printisol** quickly and opaquely to the clean plastic model with the enclosed brush.
4. Then allow to dry for 2 minutes. Repeat the process for print layer thicknesses of > 100 µm.
5. Process the cold-curing polymer according to the manufacturer's instructions.
6. Immediately after polymerization, carefully lift the cold-curing resin from the 3D model using an instrument.
7. Carefully remove wax and insulation residues with a steam jet.

5. Notice


Only suitable for extraoral use. 3D models consist of individual print layers. The larger the print layer, the stronger the mechanical retentions created. These undercut areas may have to be blocked out with a layer of wax.

Attention: The batch number and the best-before date are indicated on each package of the material. If you have a complaint, please always quote the batch number of the product. Do not use the product after the best-before date.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. Do not smoke. Keep container tightly closed. Ground container and system to be filled. Use explosion-proof electrical/ventilation/lighting equipment. Use non-sparking tools. Take measures against electrostatic discharges. Avoid inhalation of dust/fume/gas/mist/vapor/aerosol. Use only outdoors or in well ventilated areas. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: Immediately call a POISON CENTER/doctor. IF ON SKIN: Wash with plenty of water. IF ON SKIN (or hair): Remove all contaminated clothing immediately. Wash skin with water. IF INHALED: Remove person to fresh air and keep comfortable for breathing. DO NOT induce vomiting. If skin irritation occurs: Get medical advice/attention. Remove contaminated clothing and wash before reuse. In case of fire: Use ABC powder to extinguish. Absorb spillage. Store in a well-ventilated place. Keep container tightly closed. Store in a well-ventilated place. Keep cool. Store under lock and key.

6. Hazard statements

Highly flammable liquid and vapor. May be fatal if swallowed and enters airways. Causes skin irritation. May cause drowsiness and dizziness. Very toxic to aquatic life with long lasting effects.

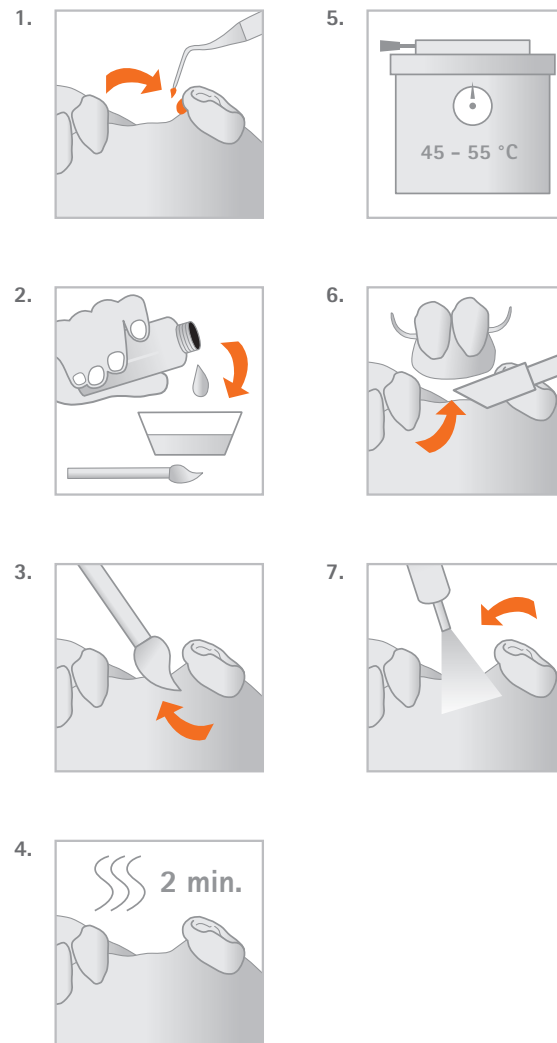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