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Distributor (US):
pro3dure medical LLC
9825 Valley View Road · Eden Prairie, MN 55344
Phone: 952-426-1928

info@pro3dure.com · www.pro3dure.com · Made in Germany

SysTherm[®] cast 2.0

Generative Resin

Instructions for use

GB 1. Intended use / Indication

US Methacrylate based resin for 3D-printing systems (DLP) with irradiation sources (385 nm, 405 nm) for the production of cast forms for PU systems.

2. Contraindications

SysTherm[®] cast 2.0 is contraindicated ...

1. ... if a user is known to be allergic to any of the ingredients.
2. ... for direct contact with uncured or partially cured material.
3. ... for any application that is not part of the indication (see above).

3. Intended users

Earmold laboratory, hearing aid acoustician

4. Requirements

Software – Information available from:

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Hardware (3D Printing) – Information available from:

ASIGA - 2/19-21 Bourke Road, Alexandria, NSW 2015, Australia

Hardware (Post Curing) – Information available from:

NK Optik GmbH (otoflash G171) · Isarstr. 2, D-82065 Baierbrunn, Germany

Additional specifications on www.pro3dure.com.

5. Material

SysTherm[®] cast 2.0 consists of functional (meth)acrylic resins, initiators, dyes and stabilisers.

6. Geometric presetting

Minimum wall thickness: 0.5 mm

7. Material parameters

Radiation penetration depth controllable by exposure time
50 µm
100 µm

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- 1) SysTherm[®] cast 2.0 bottles should be shaken well before use. Ensure that SysTherm[®] cast 2.0 is tempered to 23 °C to 30 °C before use.
- 2) Carefully pour SysTherm[®] cast 2.0 into the specified printer tray of the production system.
- 3) Ensure that the correct material parameters are selected on the printer.
- 4) Print molds from SysTherm[®] cast 2.0. Ensure that the vent holes are level with the filler neck. Direct post-processing is recommended after completion of the build process. If this cannot be guaranteed, leave the produced objects in the printing system.
- 5) After printing, clean the parts thoroughly with IPA in a 5-minute ultrasonic bath and then rinse them with fresh IPA.
- 6) Dry the parts thoroughly inside and out with compressed air.

7) Fill the molds with SysTherm[®] SPT-1 or SPT-2 separator. Drain off any excess liquid. Ensure that all surfaces are wetted. Remove all excess liquid.

8) Dry the parts thoroughly inside and out with compressed air.

9) Curing with Otofash G171: Cure the parts with 2 x 1000 flashes in an Otofash G171, under nitrogen. Or ...

10.1) Curing with CD-2 from pro3dure: Dry the parts in an oven at 40 °C for at least 30 minutes to remove any residual solvent.

10.2) Then cure the parts for 5 minutes in the CD-2 under nitrogen. **Note:** The separator remains on the surface as a thin film of liquid.

9. Notice

The batch number and best-before date are indicated on each package. In case of complaints, please always state the batch number of the product. Do not use the product after the expiration date. Deviations from the described manufacturing processes or storage conditions may lead to changes in the mechanical and optical properties of the material. Due to its chemical base, SysTherm[®] cast 2.0 is sensitive to moisture and should be stored in a dry place at room temperature. In accordance with the EU Medical Devices Regulation, users/patients are obliged to report serious incidents involving a medical device to the manufacturer and the competent authority of the country in which they occurred.

10. Hazard Statements:

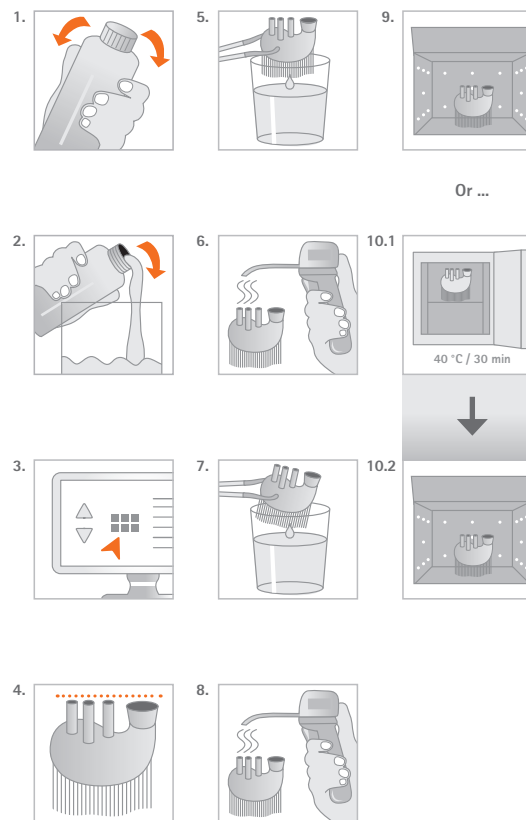
Causes skin irritation. May cause an allergic skin reaction. Causes serious eye irritation. May cause respiratory irritation. May damage the unborn child. Very toxic to aquatic life with long lasting effects.

11. Precautionary Statements:

Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Avoid breathing dust/fume/gas/mist/vapours/spray. Avoid contact during pregnancy/while nursing. Wash hands thoroughly after handling. Use only outdoors or in a well-ventilated area. Contaminated work clothing should not be allowed out of the workplace. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF exposed or concerned: Get medical advice/attention. If skin irritation occurs: Get medical advice/attention. If skin irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash it before reuse. Collect spillage. Store in a well-ventilated place. Keep container tightly closed. Store in a closed container. Store locked up.

pro3dure
medical

Materials are our DNA!

Or ...

Product compatibility

	SysTherm [®] ECO	SysTherm [®]
SysTherm [®] cast 2.0	SysTherm [®] SPT-1 or SPT-2	SysTherm [®] SPT-1 or SPT-2

Physical properties*:

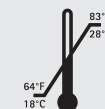
SysTherm[®] cast 2.0

■ Density:
approx. 1.1 g/ml

■ Viscosity:
approx. 0,5 Pa s

■ Cured material:
(depending on irradiation unit)
Modulus of elasticity:
approx. 1000 MPa
Flexural strength:
approx. 50 MPa
Elongation at break:
approx. 10 %
Hardness:
ca. 80 Shore D

■ Storage:

* These data come from measurements of a representative sample that was determined as part of our quality assurance.

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pro3dure medical GmbH