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**Distributor (US):**  
pro3dure Medical LLC  
9825 Valley View Road · Eden Prairie, MN 55344  
Phone: 952-426-1928

info@pro3dure.com · www.pro3dure.com · Made in Germany

- 1. Intended use / Indication**  
Polyurethane material for the production of earmolds.
- 2. Contraindication**  
SysTherm<sup>®</sup> ECO is contraindicated ...  
1. ... if a patient is known to be allergic to any of the ingredients.  
2. ... for direct contact with uncured or partially cured material.  
3. ... for any application that is not part of the indication (see above).

- 6) Carefully remove the SysTherm<sup>®</sup> earmold from the silicone or cast mold. Wear safety goggles and use suitable tools. Clean and process the SysTherm<sup>®</sup> ECO earmolds. Avoid aggressive solvents such as acetone or isopropanol.

- 9. Post-processing**  
Remove the raw earmold from the silicone or 3D-printed cast mold. It can then be finished with a carbide burr. The earmold can then be sanded with sandpaper or processed using a vibratory finishing process.

- 10. Cleaning**  
Clean the finished earmold with soapy water and then dry it with a paper towel.

- 11. Cleaning and care**  
After wearing your earmold, rinse it with warm soapy water and clean all surfaces with a soft paper towel. Store the SysTherm<sup>®</sup> ECO earmold in a dry box for hearing aids.

- 12. Cleaning agents and conditions to avoid**
- toothpaste or commercially available denture cleaning agents
  - alcohol-based cleaning agents or mouthwashes
  - abrasive cleaners or general household cleaners
  - hot water (> 60 °C) or ultrasonic cleaning devices


- 13. Component A**  
**Hazard Statements:**  
May cause an allergic skin reaction. Harmful if inhaled. May cause respiratory irritation. Toxic to aquatic life with long lasting effects.

- Precautionary Statements:**  
Avoid breathing dust/fume/gas/mist/vapours/spray. Use only outdoors or in a well-ventilated area. Contaminated work clothing should not be allowed out of the workplace. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. IF INHALED: Remove person to fresh air and keep comfortable for breathing. Get medical advice/attention if you feel unwell. If skin irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash it before reuse. Collect spillage. Store in a well-ventilated place. Keep container tightly closed. Store locked up. Dispose of contents/container according to official regulations.

- 14. Component B**  
**Hazard Statements:**  
Causes serious eye irritation. Suspected of damaging fertility or the unborn child.

- Precautionary Statements:**  
Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Avoid contact during pregnancy/while nursing. Wash hands thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF exposed or concerned: Get medical advice/attention. Store locked up. Dispose of contents/container according to official regulations.

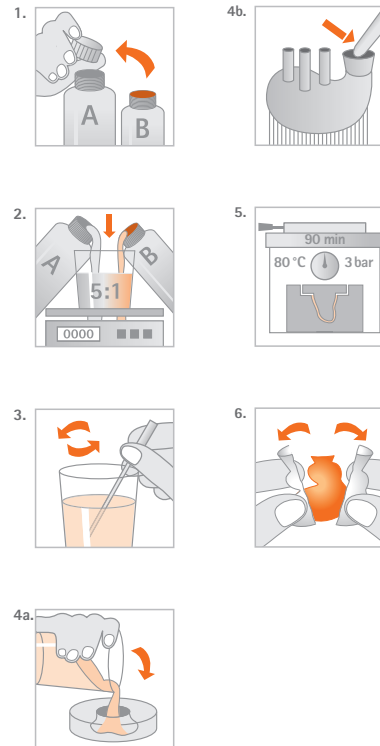UMDNS 16-697

### Product Compatibility

	SysTherm <sup>®</sup> ECO
audioprint <sup>®</sup> GR-12.1	SysTherm <sup>®</sup> SPT-2
audioprint <sup>®</sup> GR-12.2	SysTherm <sup>®</sup> SPT-2
audioprint <sup>®</sup> GR-12.3	SysTherm <sup>®</sup> SPT-2
SysTherm <sup>®</sup> cast 2.0	SysTherm <sup>®</sup> SPT-1 or SPT-2

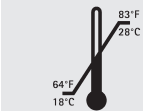
Physical properties\*: Ordering information:

SysTherm<sup>®</sup> ECO

SysTherm<sup>®</sup> ECO

- Setting time:  
90 min. at  
80 °C / 176 °F  
in pressure pot  
at 3 bar
- Final hardness:  
Shore A  
at 23 °C: 80-95  
Shore A  
at 37 °C: 65-75
- Application:  
at 23 ± 2 °C,  
50 ± 5 %  
relative humidity
- Storage:

- A-component**  
1000 g bottle  
REF: A4020250
- B-component**  
clear-transparent  
200 g bottle  
REF: A4020252
- rose-transparent**  
200 g Flasche  
REF: A4020254

- A-component**  
500 g bottle  
REF: A4020251
- B-component**  
clear-transparent  
100 g bottle  
REF: A4020253
- rose-transparent**  
100 g bottle  
REF: A4020255

\* These data come from measurements of a representative sample that was determined as part of our quality assurance.

This side up.  
Diese Seite nach oben.  
Ce côté vers le haut.  
Este lado hacia arriba.  
Questo lato in su.

Temperature limit  
Temperaturgrenze  
Limite de température  
Limite de temperatura  
Limite di temperatura

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**Distributor (US):**  
pro3dure medical LLC  
9825 Valley View Road · Eden Prairie, MN 55344  
Phone: 952-426-1928

info@pro3dure.com · www.pro3dure.com · Made in Germany

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pro3dure medical GmbH