1. Intended use / Indication

1-component lacquer for surface sealing of hearing protections.

2. Contraindication

- ... if a patient is known to be allergic to any of the ingredients.
- ... for every application that is not part of the indication (see above).

3. Product Description

The **audioprint® L-2 FLX | MSI** lacquer includes a lactam-based, biomimetic solution for multi-species inhibition*. This reduces the formation of undesirable biofilms on audio appliances without a microcidal effect.

*c. albicans / s. mutans

4. Patient Target Group

Persons for whom a hearing protection is to be created.

5. Intended Users

Earmold laboratory, ENT specialist, hearing aid acoustician

6. Application

Unscrew cap of the pro3dure single component lacquer **audioprint® L-2 FLX | MSI** container and pour into a dipping jar (see fig. 1). The pro3dure single component lacquer **audioprint® L-2 FLX | MSI** can be brushed or dipped onto the surface earmolds or shells directly (see fig. 2). After coating earmolds or shells place parts in a pro3dure **CD-2** curing unit for complete curing of the pro3dure single component lacquer **audioprint® L-2 FLX | MSI** (see fig. 3) or any other appropriate curing device unit.

7. Important working hints

A careful preparation of the shell or earmold, well insure proper adhesion of the pro3dure single component lacquer **audioprint® L-2 FLX | MSI**, a thorough cleaning will ensure reliable results. Always place cap back on pro3dure single component lacquer **audioprint® L-2 FLX | MSI** when finished pouring in dipping jar. Make sure pro3dure single component lacquer **audioprint® L-2 FLX | MSI** is properly cured to insure biocompatibility. We recommend standard gloves made of nitrile or polyethylene.

8. Notice

To avoid detrimental effects on material quality do not expose the liquid material to irradiation under any circumstances. Deviations from the described manufacturing processes or storing conditions may lead to different mechanical and optical properties of the material. According to the EU Medical Devices Regulation, users/patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred.

Caution: Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Avoid breathing dust/fume/gas/mist/vapours/spray. Wash hands thoroughly after handling. Use only outdoors or in a well-ventilated area. Contaminated work clothing should not be allowed out of the workplace. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF exposed or concerned: Call a POISON CENTER/doctor. Call a POISON CENTER or doctor/physician if you feel unwell. Specific treatment (see information on this label). If skin irritation occurs: Get medical advice/attention. If skin irritation or rash occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention. Take off contaminated clothing and wash it for reuse. Store in a well-ventilated place. Keep container tightly closed. Store locked up. Dispose of contents/container according to official regulations.


9. Hazard statements

Causes skin irritation. May cause an allergic skin reaction. Causes serious eye irritation. May cause respiratory irritation. Suspected of damaging fertility.

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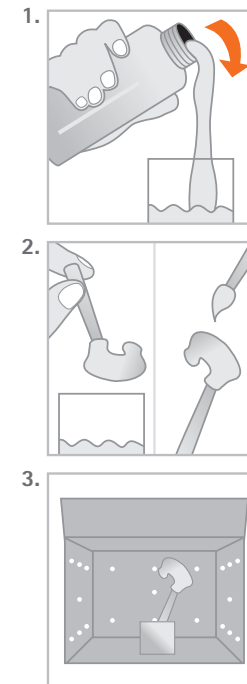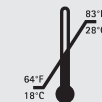
Distributor (US):
pro3dure medical LLC
9825 Valley View Road · Eden Prairie, MN 55344
Phone: 952-426-1928

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