## **Generative Resin GR-15 gingiva**

## Instructions



pro3dure medical GmbH

Am Burgberg 13 D - 58642 Iserlohn Tel. +49 (0)2374 920050-0 Fax +49 (0)2374 920050-50 info@pro3dure.com www.pro3dure.com



3.

## 1. Product description

pro3dure's generative resin GR-15 gingiva is a resin for the generative production of dental objects based on image projection systems ( $\leq$  405 nm). The formulation of GR-15 gingiva is optimized for the requirements of a robust production guaranteeing constant high quality. The material can be used for build processes with layer thicknesses from 25–100  $\mu m$ . It is recommended to use the CD-1 or CD-2 curing device from pro3dure medical for post curing.

Product description: photopolymerizable resin for the production of dental objects by image projection systems (≤ 405 nm)



- GR-15 gingiva bottles should be well shaken before use (fig 1).
- Make sure that **GR-15 gingiva** material is adjusted to temperature range 23 °C to 30 °C.
- Carefully pour **GR-15** gingiva into the vat of the image projection unit (fig 2).
- Bubbles can be removed with a cleaned spatula or by a recoater routine.
  If it is possible, always store a bottle GR-15 gingiva in your production
- unit in order to avoid temperature differences during refilling.
- For the build parameter adjustment please refer to the machine data sheet.
- After the build process is finished a direct post treatment is recommended. If this cannot be done at the time leave the produced objects in the liquid **GR-15** gingiva resin.



■ The dental objects generated out of the generative resin GR-15 gingiva can be repaired with GR-15 gingiva resin.

Impurity due to operation mistakes cannot be excluded. With respect to the low viscosity of the resin it is possible to filtrate the **GR-15 gingiva**. It is recommended to filter and stir up the resin on a regular base (fig 3). To avoid bubbles let **GR-15** gingiva rest for 1 hour before usage.



## 3. Important

- To avoid detrimental effects on material quality do not expose the liquid material to irradiation under any circumstances.
- Deviations from the described manufacturing process may lead to different mechanical and optical properties of the **GR-15** gingiva material.
- Wear personal protective gear during processing.
- Caution: Polymerised resins are chemically resistant avoid stains on clothing!
- Avoid any contact with skin and eyes. In case of accidental contact, rinse with adequate running water. Consult a doctor if necessary.
- The lot number and the expiration date are indicated on each bottle of **GR-15 gingiva**. In case of claims please always indicate the lot number of

# the product. Do not use the product after expiry of the best before date.

## Safety advice

pro3dure medical GmbH is not liable for any damages caused by improper application of the material. To be used by trained specialist personnel for the purpose indicated only.

## Technical data:

- Colour: orange red transparent
- Density: ca. 1.1 g/ml
- Viscosity: ca. 2,5 Pa s
- Post cured material: (depends on postcuring unit) ca. 60 Shore A
- Storage:



### **Ordering** information:

Standard packing:

500 g bottle, orange red transparent item no.: D1000342

1kg bottle, orange red transparent

item no.: D1000340



These data result from measurements of a representative sample, which were determined within the scope of our quality assurance.